




NEW



SOOO ARTFUL

WITH **VISALYS® FILL** & **VISALYS® FLOW**

UNIVERSAL NANO-HYBRID-FILLING-COMPOSITE

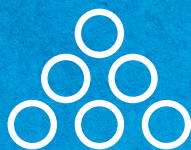

KETTENBACHDENTAL
Simply intelligent

VISALYS® FILL & VISALYS® FLOW

Light-curing universal Nano-Hybrid-Filling-composite in two consistencies

FOR ANY APPLICATION

- For all Class I to VI fillings
- For core build-ups under crowns and repairing defective fillings
- For esthetic shape correction
- Can be used with any standard light-curing bonding

FLEX SHADE SYSTEM FOR YOUR EASY SHADE SELECTION

- Choose the right shade with the aid of the Flex Shade System
- Blends harmoniously with the cavity due to the chameleon effect for **all 16 VITA shades** in the anterior and posterior area with **just 5 shades**

PLEASANT HANDLING

- Does not stick to the instrument, good modeling (Visalys® Fill)
- Perfect balance between flowability and stability (Visalys® Flow)

OUTSTANDING MECHANICAL PROPERTIES

- High filler content: 87% (Visalys® Fill) or 80% (Visalys® Flow)
- Excellent flexural strength:
147 MPa (Visalys® Fill) or 112 MPa (Visalys® Flow)

LONG FILLING DURABILITY

- Minimal discoloration due to low water absorption
- Low wear resulting from sophisticated filler technology for durable fillings



SHADE MATCHING MADE EASY DUE TO THE **FLEX SHADE SYSTEM**



Find the right shade with support of the **Flex Shade System**

The “**Flex Shade System**”, with its chameleon effect and just **5 shades (A1-A4)**, covers the entire VITA classical A1-D4 shade scale.

VISALYS® FILL & VISALYS® FLOW:

Here you will find the main colors and the additional Vita colors covered.¹⁾

A1
LIGHT BRIGHT



B1



C1

A2
BRIGHT



B2



D2

A3
MEDIUM BRIGHT



B3



D3



D4

A3.5
MEDIUM DARK



C2



C3



B4

A4
DARK



C4

SOOO esthetically pleasing...
fillings using **VISALYS® FILL** &
VISALYS® FLOW with the
Flex Shade System.




The **Bleach shade** is also available for particularly bright teeth, as well as an **OA2 opaque shade** to **conceal** discolored substrates.




¹⁾ Occlusal fillings in artificial teeth, with Visalys® Fill to demonstrate the consistency of the Flex Shade System

VISALYS® FILL & VISALYS® FLOW – DURABLE STABILITY

VISALYS® FILL Flexural strength before and after thermal alternating load²⁾

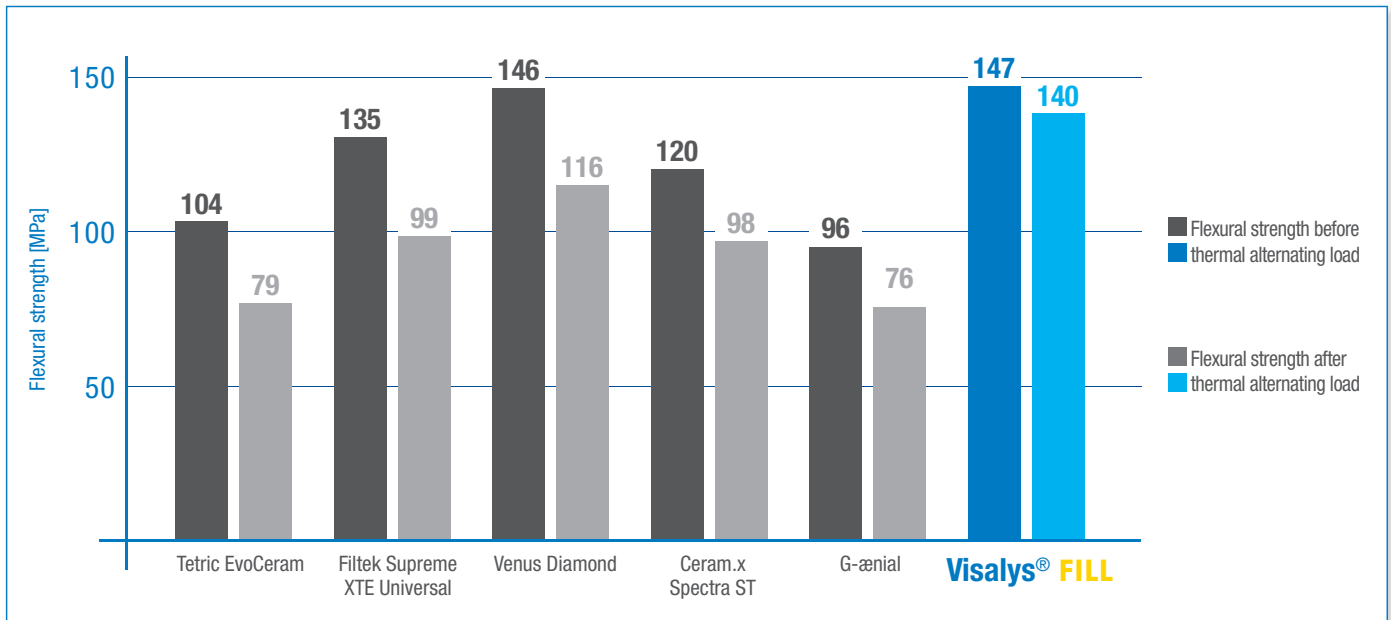

VISALYS® FLOW Flexural strength before and after thermal alternating load²⁾

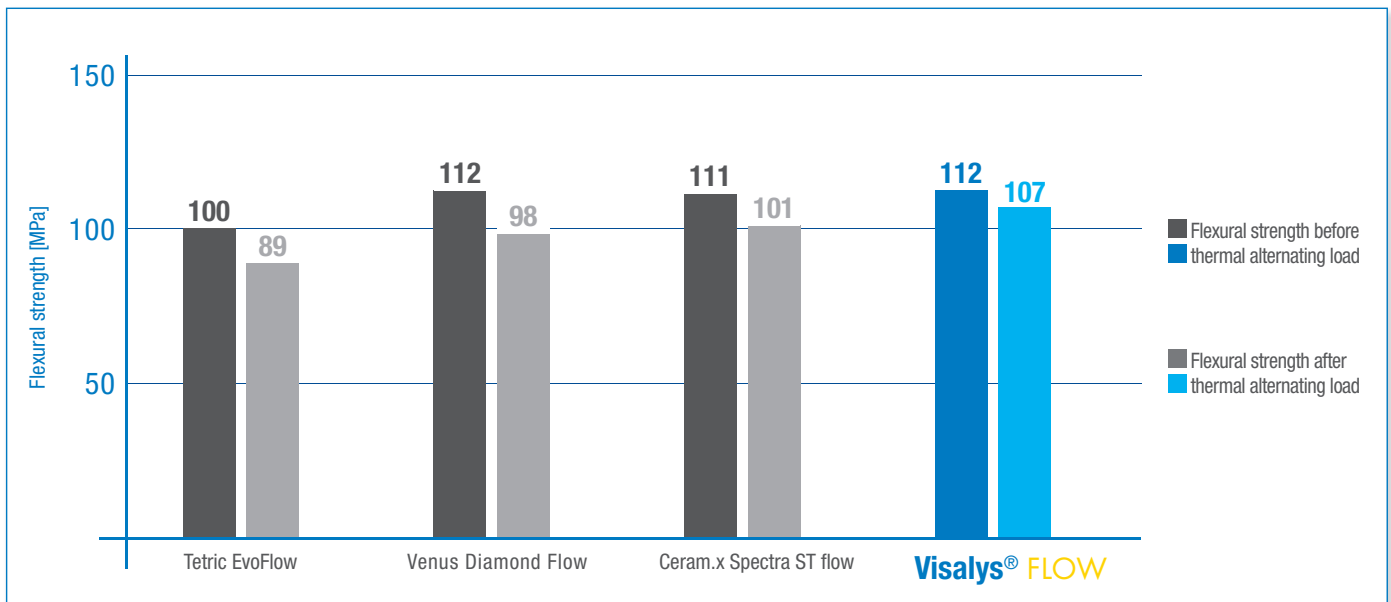


TOP FLEXURAL STRENGTH EVEN AFTER EXERTION


- Thermal load corresponds to up to 12 months in the mouth (5000 cycles at 5°C and 55°C)
- The values represent sustained stability under *in-vivo* conditions

MORE GLASS THAN PLASTIC

VISALYS® FILL Inorganic percentage in the composite³⁾

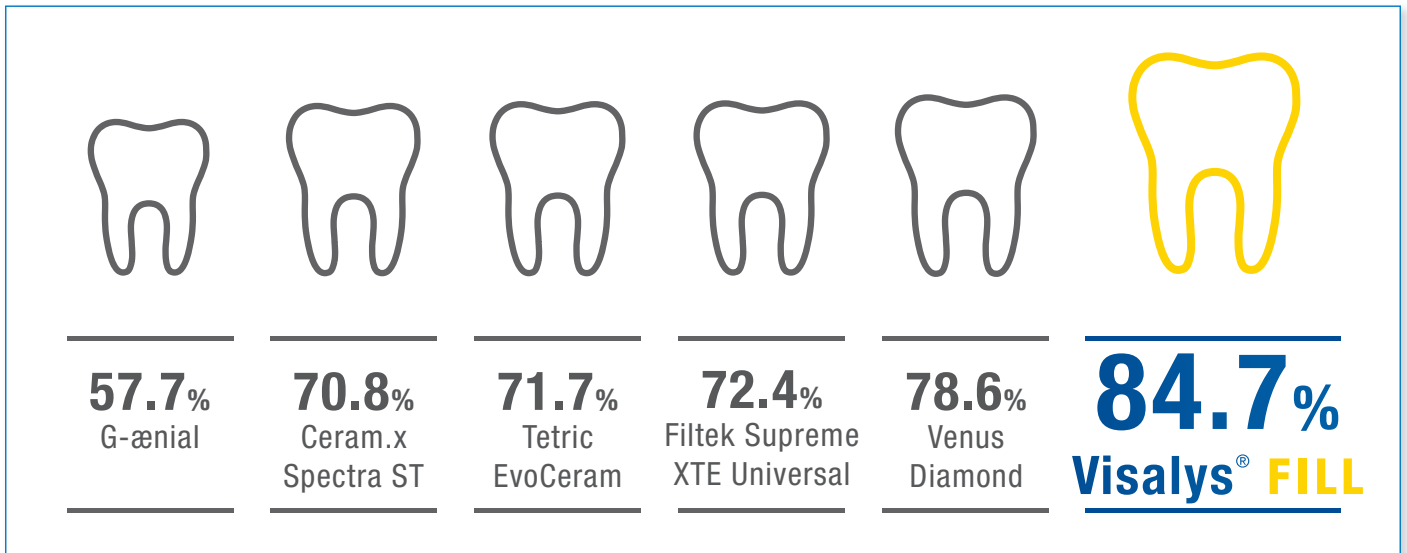

VISALYS® FLOW Inorganic percentage in the composite³⁾

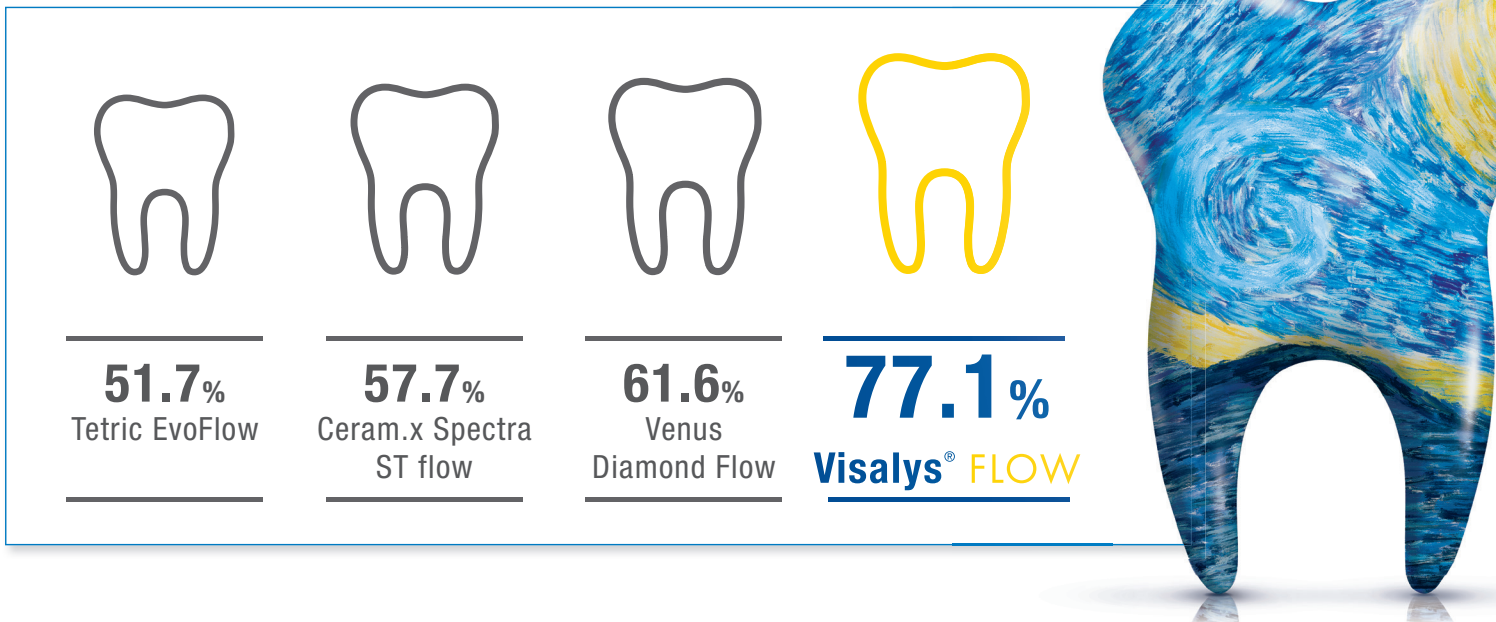


VERY HIGH INORGANIC FILLER CONTENT

- For good stability that lasts
- For excellent compatibility due to high biocompatibility


DURABLE COLOR STABILITY

VISALYS® FILL Water absorption ²⁾

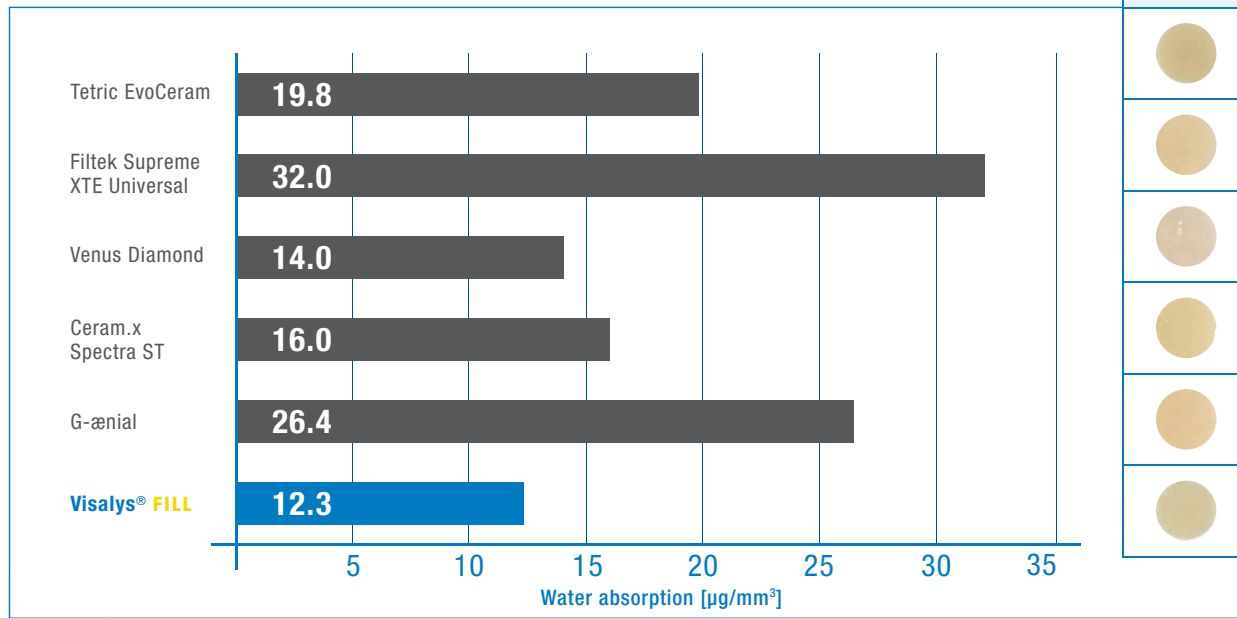

H ₂ O ⁴⁾	Coffee ⁴⁾	Wine ⁴⁾

VISALYS® FLOW Water absorption ²⁾

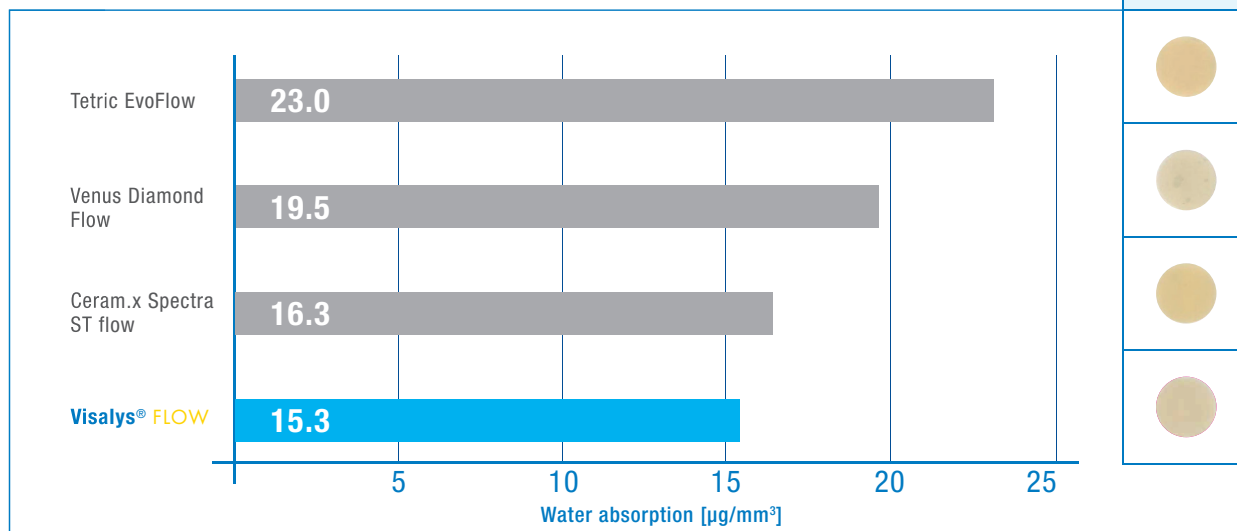

H ₂ O ⁴⁾	Coffee ⁴⁾	Wine ⁴⁾



LOW WATER ABSORPTION


- Prevents discoloration of fillings
- For long-lasting esthetics




KETTENBACHDENTAL
Simply intelligent

LOW SHRINKAGE

VISALYS® FILL volume shrinkage⁵⁾

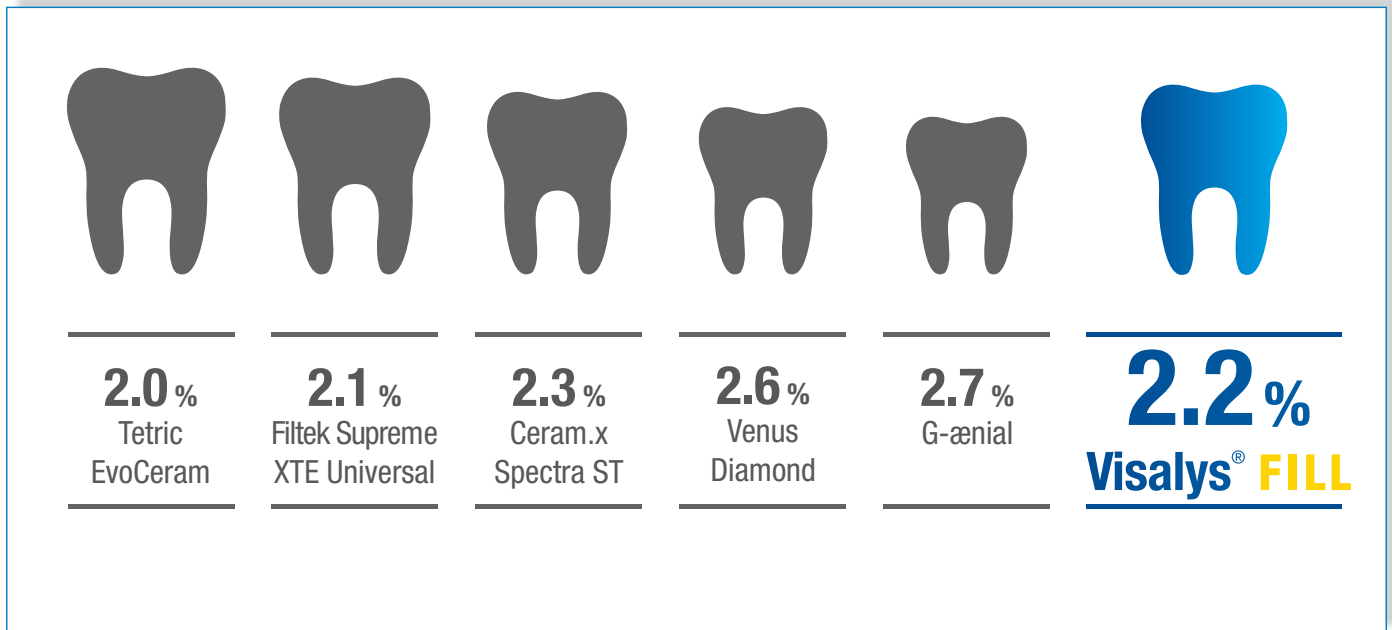

VISALYS® FLOW volume shrinkage⁵⁾

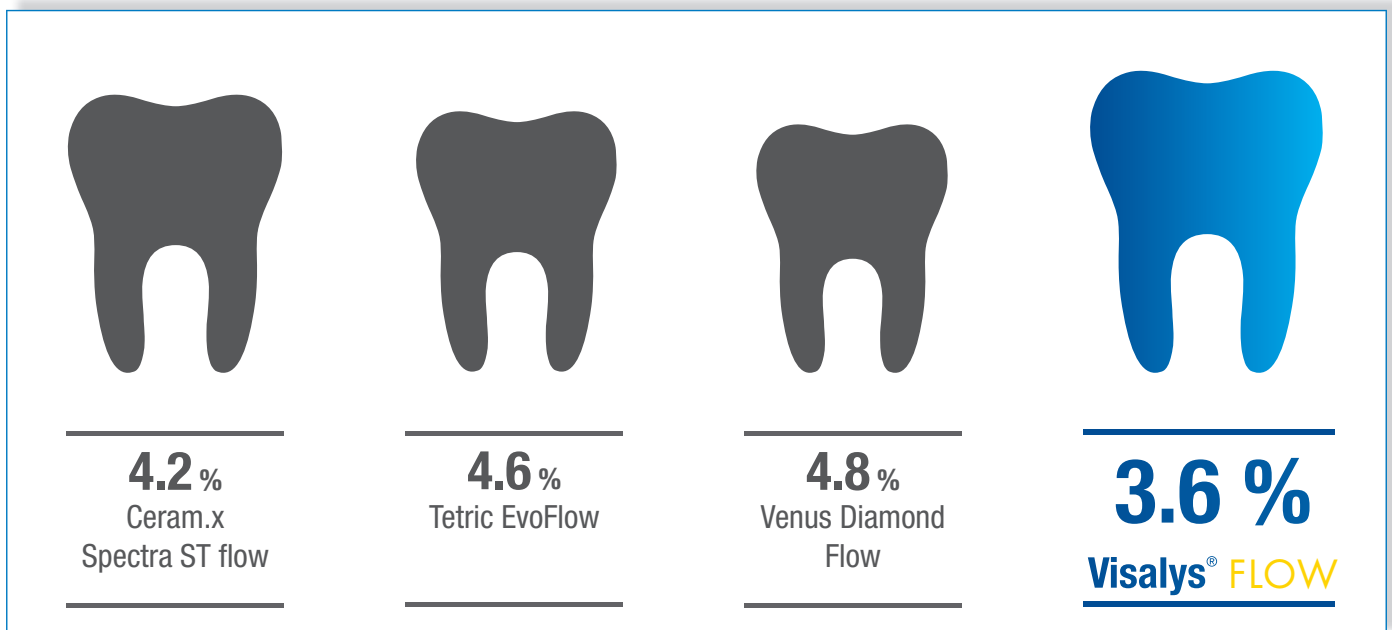

LOW SHRINKAGE

- For long lasting restorations
- For anchoring the filling in the tooth cavity
- Reduces the risk of marginal gaps and secondary caries


THE RANGE



Visalys® Fill & Visalys® Flow

Visalys® Fill

1 x 4 g syringe

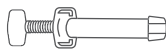

A1	REF 1500299
A2	REF 1500399
A3	REF 1500499
A3.5	REF 1500599
A4	REF 1500699
Opaque A2	REF 1500799
Bleach	REF 1500899



Visalys® Fill

15 x 0.25 g caps




A1	REF 1501299
A2	REF 1501399
A3	REF 1501499
A3.5	REF 1501599
A4	REF 1501699
Opaque A2	REF 1501799
Bleach	REF 1501899




Visalys® Flow

1 x 1.8 g syringe

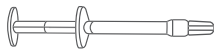

A1	REF 1503099
A2	REF 1503199
A3	REF 1503299
A3.5	REF 1503399
A4	REF 1503499
Opaque A2	REF 1503599
Bleach	REF 1503699

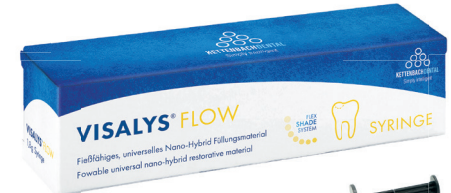

Made in Germany

NOTES

Kettenbach GmbH & Co. KG
 Im Heerfeld 7
 35713 Eschenburg · Germany
 Telefon +49 (0)2774 7050
 info@kettenbach.com
 www.kettenbach-dental.com



EN1410675_0923

